---

**Arthrosurface NanoFx® Hand Instruments**

---

**Description**

The Arthrosurface® NanoFx® instruments consist of a reusable hand instrument and a single use disposable guide wire for performing a microfracture technique for the treatment of articular cartilage defects or where bone marrow stimulation is appropriate. The optional, reusable NanoFx Thumb Tab Accessory may be used to expedite removal and repositioning of the Guide Wire.

**Instructions for Use**

Treatment using the Arthrosurface NanoFx instruments will typically be accomplished as part of an arthroscopic or minimal access surgical procedure. No specific or unique surgical incisions are required. The Arthrosurface NanoFx Guide Wire is placed tip first into the proximal lumen of the Arthrosurface NanoFx Hand Instrument. The distal tip of the instrument is then positioned at the target site. A light mallet strike on the exposed proximal tailstock of the guide wire is sufficient to drive the wire to its full depth. The NanoFx Hand Instrument is then removed and repositioned to create additional penetration sites. The optional, reusable NanoFx Thumb Tab Accessory may be used to expedite removal and repositioning of the Guide Wire.

**Warnings and Precautions**

Arthrosurface NanoFx Hand Instruments and Thumb Tab Accessories are provided **NON-STERILE**. They must be properly cleaned and sterilized before each use. The surgeon shall be thoroughly familiar with the instruments and surgical technique prior to performing the procedure. The Arthrosurface NanoFx Hand Instruments and Thumb Tab Accessories should be regularly inspected for any signs of wear or damage.

Arthrosurface NanoFx Hand Instruments are designed to be used with Arthrosurface accompanying NanoFx Guide Wires and optional NanoFx Thumb Tab Accessory. Use of guide wires from different manufacturers may create patient safety issues.

Guide wires are to be driven with mallet strike only.  
**Do not drive with drill or powered handpiece.**

**Cleaning**

Cleaning by hand rather than by mechanical cleaning will prolong the life of the instruments. Clean all crevices, flutes, and cannulations of all debris, using a soft bristle brush or cleaning stylet. Remove all traces of blood or other residues immediately. Do not allow these to dry. Instruments should be cleaned while submerged in warm water with an appropriate neutral pH detergent. Always follow the manufacturer's instructions when preparing and using detergents. Do not use steel brushes as they can accelerate wear and corrosion of the instruments. Rinse instruments thoroughly with distilled water. Dry instruments immediately after cleaning.

**Possible Adverse Effects**

Complications reported with microfracture surgery include general surgical complications (infection, blood clot, incisional irritation). Complications specific to the microfracture technique are poor tissue differentiation or repair, and osteophyte formation.

**Sterilization**

Recommended parameters for steam sterilization are as follows:

Cycle	Temperature	Minimum Exposure Time
Vacuum	270° F/ 132° C	4 minutes
Gravity	250° F/ 121° C	30 minutes

Recommended dry time is 30 minutes.

Sterilizing in liquid solutions is not recommended. Do not sterilize at temperatures greater than 275° F/ 135° C.

**Caution**

Federal Law (USA) restricts this device to sale by or on the order of a physician.

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© 2021 ArthroSurface, Inc. All rights reserved.
ArthroSurface, Inc.
MT Promedt Consulting GmbH
Printed in the U.S.A.